

## “Accufit” MRI Coil Drape Instructions For

### Model # BSM1007-01 and # BSM1008-01

The “AccuFit” MRI Coil drape is designed to protect your equipment and patients from any oils, dirt or contact skin issues. This coil drape is compatible with the coils listed and may fit other models:

Siemens 1.5T Avanto/Espree 7 Channel  
Philips 1.5T Achieva 7 Channel  
GE 8 Channel coil  
GE 3.0T 7 Channel Breast  
May fit coils other than those listed

Siemens 1.5T Symphony 7 Channel  
Philips 3.0T SENSE  
GE 1.5T Excite 7 Channel  
CP Siemens Breast Array



Special features are designed into this drape. Occasionally due to lesion accessibility, the coil pad is removed. To accommodate this situation the Coil cover is lined with extra fabric for patient comfort.

**1.** Open the package. The top of the drape will align the round holes to match the coil holes.



**2.** Pull elastic around all edges for a secure fit.

**NOTE:**  
The inner padding allows the coil pad to be removed and the AccuFit™ Coil drape to be utilized. The padding faces down next to the coil.

